# CareMat

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# CareMat

## with 868 MHz EasyWave® radio transmitter

The pressure-sensitive bed mat for dementia patients and persons at risk of falling

### Translation of the original instructions

## 1 Safety instructions

- Read these operating instructions thoroughly before putting the device into operation and keep them for future reference.
- Not a safety component in accordance with the EU Machinery Directive must not be used for personal protection or EMERGENCY STOP function.
- Do not use this product other than for its specified application.
- Failure to follow these safety precautions may cause damage to device or objects, serious personal injury, or death.
- Observe all applicable local, national, and international safety standards, codes, and laws.
- After accessing the inside of the device, ensure the cover/protection seal is closed tightly to achieve designated protection rating.
- Warranty claims will be invalid in the event of any damage to the product, to goods or persons that is due to failure to observe these instructions.

### Instructions for nursing staff

- The CareMat operates purely as a support for the nursing staff, but is not a substitute for them.
- The functionality of the CareMat is to be checked each time that it is used.

### Notes for the installer

- This CareMat may only be operated together with compatible radio receivers with EasyWave® coding, manufactured by ELDAT GmbH.
- The electrical installation work and commissioning of the product may only be carried out by a qualified technician.
- Avoid touching any electronic components.
- ESD safety rules should be observed.

## 2 Product description

### 2.1 Radio system description

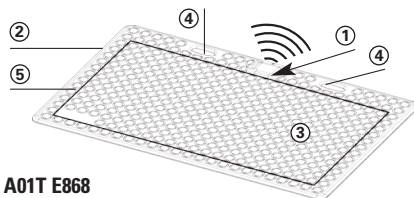
The CareMat with 868 MHz EasyWave® radio transmitter is operated together with compatible radio receivers with EasyWave® coding, manufactured by ELDAT GmbH. In care facilities, the radio receivers are usually paired with an existing call system infrastructure. When the mat is activated, the call system infrastructure emits an alarm signal.


### 2.2 Areas of application and function

The CareMat was primarily designed for use in retirement and nursing homes, hospitals and psychiatric wards as well as for sheltered residential accommodation. The necessary care and supervision of patients with dementia and of those who are at risk of falling represents a significant increase in care expenditure for care practitioners. The CareMat is the ideal support: risks of falling and the dangers which arise from wandering or loss of orientation can be significantly reduced.


The CareMat as a pressure-sensitive sensing device is laid directly in front of the care bed or the door to the room. As soon as the person leaves the bed and steps onto the CareMat, this switches on and sends out an alarm signal to the call system via the radio receiver. The wireless signalling system reduces the risk of falling caused by wiring on the floor.

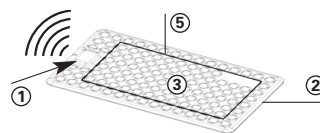**CareMat A01T E868**  
**CareMat A11T E868**  
1100 x 700 mm

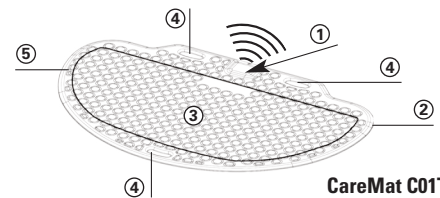**CareMat B01T E868**  
**CareMat B11T E868**  
700 x 400 mm


**CareMat C01T E868**  
**CareMat C11T E868**  
1100 x 700 mm,  
half round

- ① Housing with radio transmitter
- ② Chamfered, wheelchair-accessible edges
- ③ Non-slip surface with stud texture design


- ④ Handles
- ⑤ Switching active zone
- ⑥ Radio receiver: NOT included

## 3 Day-to-day operation

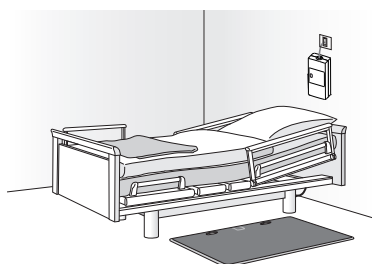Align the CareMat centrally in front of the bed with the handles underneath the edge of the bed so that missing the mat is impossible.

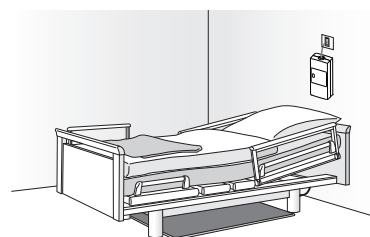EasyWave® radio receiver paired with the call system (observe radio receiver operating instructions)

Should the CareMat not be needed, it can be pushed underneath the bed.

## 4 Cleaning and disinfecting

- The CareMat can be cleaned using commercially available disinfecting and cleaning agents with max. 70 vol% alcohol. No corrosive agents should be used.
- The CareMat is neither autoclavable nor can it be disinfected using steam.

## 5 Transport and storage

- During transport and storage, any significant bending of the CareMat should be avoided.
- Store the CareMat in a dry, clean and flat location at 0 °C to +55 °C.
- Do not place any heavy loads on the CareMat and do not stack more than 10 mats on top of each other.

## 6 Commissioning and maintenance

### 6.1 Configuration / Teaching in the transmitter

Commissioning the CareMat with 868 MHz EasyWave® radio transmitter requires the use of a corresponding radio receiver with EasyWave® coding. As the main functions of this transmitter/receiver system are executed in the radio receiver, please consult the radio receiver operating instructions to find the steps required for commissioning.

For information on how to teach the CareMat into a radio receiver, consult the appropriate operating instructions; this information can be found in chapters covering programming or teaching in transmitters, for example. If the instructions state that the transmitter button needs to be activated, what this is referring to in the case of the CareMat with 868 MHz EasyWave® radio transmitter is activating the CareMat. The CareMat is activated when it is stepped on or when a hand is pressed forcefully onto the active switching surface.

The radio receiver operating instructions also explain how to connect radio receivers to existing call systems. As well as this, they describe various functions and operation modes such as the display indicating that the battery for the radio transmitter inside the CareMat is low.

### 6.2 Maintenance and changing the radio transmitter battery

Aside from changing the battery, the CareMat with 868 MHz EasyWave® radio transmitter is maintenance-free. To change the battery, proceed as follows:

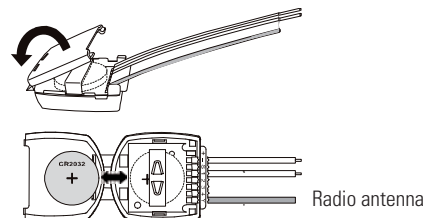
- Remove the housing cover on the rear of the CareMat (6 countersunk screws M3 x 5, cross-head size PH1).
- Carefully lift the transmitter housing off the CareMat housing. Do not unplug the cable connections.
- Lift the transmitter housing cover as shown in the illustration. Remove the spent battery and insert a new one (type CR2032, 3 V lithium). Close the transmitter housing cover.
- Insert the transmitter housing back in the CareMat housing and carefully place the connection cables and the radio antenna cable back in the CareMat housing.
- Reinstall the housing cover. Carefully tighten the 6 screws by hand; do not overtighten.

For more details on 868 MHz EasyWave® radio receivers, please refer to the Eldat website at [www.eldat.com](http://www.eldat.com).

#### Note

Once the CareMat has been paired with a radio receiver, these components form a call system. Especially in cases where multiple CareMat call systems are being used, we highly recommend labelling components that belong together to indicate this once teach-in is complete. In this way, it can be ensured that the components are not mixed up and that the call is only triggered in the particular room where the CareMat and the associated radio receiver are also actually located.

When performing a battery change, ensure that the battery does not come into contact with the soldered joints of the cables – risk of short circuit. Locate the battery centrally.

## 7 Fault finding

If the system functions are not being provided, the user should check the following:

- Has another transmitter/receiver system been paired, or have any components been mixed up?
- Is the CareMat's radio transmitter battery OK?
- Has the information in the operating instructions for the radio receiver been observed?

- If the system is being operated together with a call system: Is the radio transmitter connected to the call system as specified?

If the points mentioned above have been checked out, but the system functions are, despite this, still not being provided, you should consult a qualified expert.

## 8 Technical data

<b>Article</b>	CareMat Ax1T E868 / Bx1T E868 / Cx1T E868	<b>Signalling protocol</b>	ELDAT EasyWave®
<b>Dimensions (LxWxH)</b>	1100 x 700 x 9 mm / 700 x 400 x 9 mm / 1100 x 700 x 9 mm, height with housing 15 mm	<b>Radio frequency</b>	868.30 MHz
<b>Weight [kg]</b>	8.2 kg / 3.2 kg / 6.5 kg	<b>Working life</b>	Dependent on application approx. 2 years
<b>Material</b>	Polyurethane (PU)	<b>Radio range</b>	Up to 30 m in buildings, dependent on structural building conditions (up to 100 m in free field)
<b>Colour</b>	Careyellow, Caregrey	<b>Type of battery</b>	1 x CR2032, 3 V lithium
<b>Surface</b>	Stud texture design	<b>Type of protection</b>	IP54
<b>Weight of persons</b>	At least 10 kg	<b>Ambient temperature</b>	0 up to +55 °C
<b>Chemical resistance</b>	Water (room temp.), body fluids, commercially available disinfectants, max. 70 vol% alcohol	<i>Subject to technical modifications</i>	

## 9 EU Declaration of Conformity

See attachment

## 10 WEEE

Devices with this symbol must be treated separately during disposal. This must be done in accordance with the laws of the respective countries for environmentally sound disposal, processing and recycling of electrical and electronic equipment.

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BBC Bircher Smart Access, BBC Bircher AG, Wiesengasse 20, CH-8222 Beringen, [www.bircher.com](http://www.bircher.com)