# CareMat

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Designed in Switzerland / Made in Switzerland

# CareMat

with 433 MHz VarioRec® radio transmitter

Pressure-sensitive bedside mat  
to detect patients who wander or fall

## Translation of the original instructions

### 1 Safety instructions

- Read these operating instructions thoroughly before putting the device into operation and keep them for future reference.
- Not a safety component in accordance with the EU Machinery Directive must not be used for personal protection or EMERGENCY STOP function.
- Do not use this product other than for its specified application.
- Failure to follow these safety precautions may cause damage to device or objects, serious personal injury, or death.
- Observe all applicable local, national, and international safety standards, codes, and laws.
- After accessing the inside of the device, ensure the cover/protection seal is closed tightly to achieve designated protection rating.
- Warranty claims will be invalid in the event of any damage to the product, to goods or persons that is due to failure to observe these instructions.

#### Instructions for nursing staff

- The CareMat operates purely as a support for the nursing staff, but is not a substitute for them.
- The functionality of the CareMat is to be checked each time that it is used.


#### Notes for the installer

- The electrical installation work and commissioning of the product may only be carried out by a qualified technician.
- Avoid touching any electronic components.
- ESD safety rules should be observed.

### 2 Product description


#### 2.1 Radio system description

The CareMat with 433 MHz VarioRec® radio transmitter is compatible with the VarioRec® radio receivers from Lehmann electronic. VarioRec® radio receivers can receive radio signals from multiple VarioRec® radio transmitters and relay them to a cable connected call system. Detailed information about this device and other compatible 433 MHz VarioRec® devices can be found on the following website: [www.lehmannweb.de](http://www.lehmannweb.de)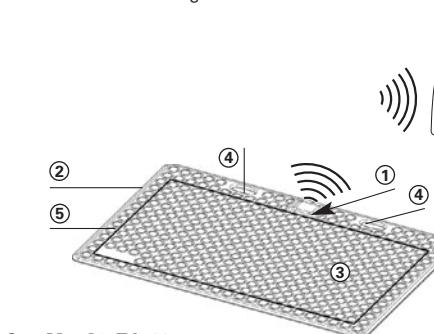


**CareMat A01T L433**  
**CareMat A11T L433**  
1100 x 700 mm

- ① Housing with radio transmitter
- ② Chamfered, wheelchair-accessible edges
- ③ Non-slip surface with stud texture design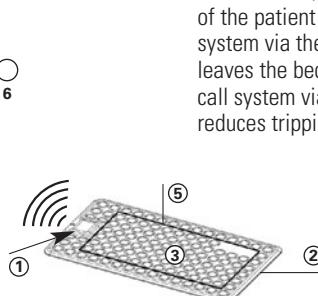


**CareMat B01T L433**  
**CareMat B11T L433**  
700 x 400 mm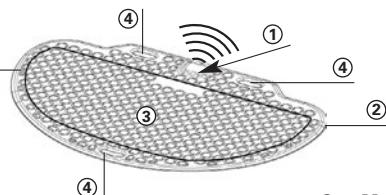

**CareMat C01T L433**  
**CareMat C11T L433**  
1100 x 700 mm,  
semi-circular


- ④ Carrying handle
- ⑤ Active switching zone
- ⑥ VarioRec® radio receiver (optional component)

### 3 Day to day operation

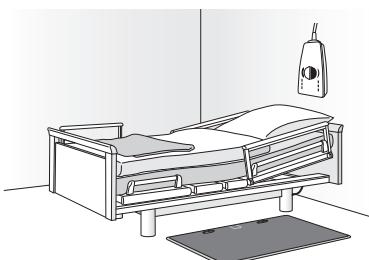CareMat should be centrally aligned next to the bed so it is impossible to bypass.

Connect the VarioRec® radio receiver to the call system.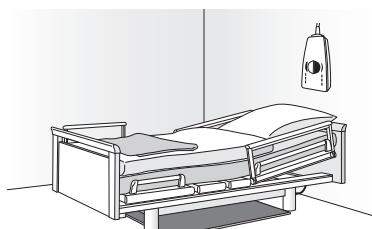

If the CareMat is not needed during the day, it can be stored underneath the bed.

### 4 Cleaning and disinfecting

- The CareMat can be cleaned using commercially available disinfecting and cleaning agents with max. 70 vol% alcohol.  
No corrosive agents should be used.
- The CareMat is neither autoclavable nor can it be disinfected using steam.

### 5 Transport and storage

- Do not bend CareMat during transport, storage or installation
- CareMat must be stored flat in a clean, dry location with temperature range 0 °C to +55 °C.
- Do not place any heavy loads on the CareMat and do not stack more than 10 packed mats on top of each other.

## 6 Radio system combined with cable-connected light signalling system

### 6.1 CareMat configuration

The CareMat with the integrated VarioRec® radio transmitter can be combined with multiple VarioRec® radio receivers of the same frequency.


Configuration and the respectively provided features can be found in the technical manuals. These can be found online at: [www.lehmannweb.de](http://www.lehmannweb.de).

The CareMat and one VarioRec® receiver are to be installed one per room. The electrical connections should be performed by a qualified technician. One CareMat and one VarioRec® radio receiver are installed per room. If anyone steps on an assigned CareMat, an alarm signal is transmitted to the VarioRec® radio receiver.

**⚠ Note:** Together, the CareMat and radio transmitter form one call system. Components forming individual systems should be labeled clearly so the call is triggered in the particular room where the CareMat and VarioRec® receiver are located.

### 6.2 Replacing radio transmitter battery

Remove housing cover on rear  
(6 countersunk screws M3x5, cross-head size PH1)

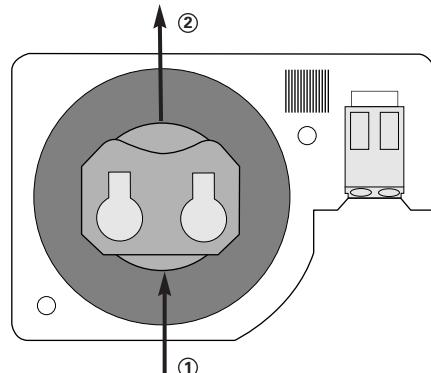- ① With a thin, pointed object (preferably plastic), push the battery out of the housing compartment.
- ② Insert the new battery, ensuring the positive (+) side is facing you

Reinstall housing cover, carefully tighten the 6 screws by hand, do not overtighten.

## 7 Maintenance and troubleshooting

The CareMat is maintenance free until the batteries need to be replaced. Ensure battery compartment screws are not over-tightened.

If batteries have been replaced and the CareMat is still not functioning properly, check the following:

- Is the call system operating and the VarioRec® radio receiver correctly connected?
- Is the CareMat correctly tuned-in to the VarioRec® radio receiver?
- Is the signal transmitter battery OK?

If the CareMat is still not functioning after the above have been checked, consult a qualified expert.

## 8 Technical specifications

<b>Article</b>	CareMat Ax1T L433 / Bx1T L433 / Cx1T L433	<b>Signaling protocol</b>	Lehmann electronic VarioRec®
<b>Dimensions (LxWxH)</b>	1100x700x9 mm / 700x400x9 mm / 1100x700x9 mm, height with housing 15 mm	<b>Radio frequency</b>	433.92 MHz
<b>Weight [kg]</b>	8.2 kg / 3.2 kg / 6.5 kg	<b>Working life</b>	Dependent on application approx. 2 – 3 years
<b>Material</b>	Polyurethane (PU)	<b>Radio range</b>	up to 30 m in buildings, dependent on structural building conditions (up to 100 m in free field)
<b>Colour</b>	Careyellow, Caregrey	<b>Type of battery</b>	1x CR2032, 3 V Lithium
<b>Surface</b>	Stud texture design	<b>Type of protection</b>	IP54
<b>Weight of persons</b>	At least 10 kg	<b>Ambient temperature</b>	0 to +55° C
<b>Chemical resistance</b>	Water (room temp.), body fluids, customary disinfectants, max. 70 vol% alcohol	<i>Subject to technical modifications</i>	

## 9 EU Declaration of Conformity

See attachment

## 10 WEEE

Devices with this symbol must be treated separately during disposal. This must be done in accordance with the laws of the respective countries for environmentally sound disposal, processing and recycling of electrical and electronic equipment.

## 11 Contact

**BBC Bircher Smart Access**, BBC Bircher AG, Wiesengasse 20, CH-8222 Beringen, [www.bircher.com](http://www.bircher.com)

Designed in Switzerland / Made in Switzerland

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